



## ADVANCED FILL OVERVIEW



 **PRESSURE DRESSING**



 **FLAT FOLD TAPE**



 **QUIKCLOT HEMOSTATIC DRESSING  
(GREEN OR BLACK WORKS)**



 **DECOMPRESSION NEEDLE**



 **NASOPHARYNGEAL AIRWAY**



 **CHEST SEAL**



 **TRAUMA GLOVES**

When folding excess medical component packaging, take care to not fold, crease or crush the product inside.  
Please contact Customer Service if you have questions.



**1) Insert Placement**  
To begin, place the inner Trauma Kit out with the pull tab facing as shown. Then insert Medical items.



**2) Medical Items Placement**  
Stack items as shown below

**A**  **FLAT FOLD TAPE**

 **CHEST SEAL**

 **NASOPHARYNGEAL AIRWAY**

**B**  **TRAUMA GLOVES**

**C**  **DECOMPRESSION NEEDLE**

**D**  **PRESSURE DRESSING**

 **QUIKCLOT HEMOSTATIC DRESSING**



**3) Securing Your Medical Items**  
1. Fold Trauma Kit inner pouch  
2. Insert into Pouch

**REQUIRES FOLDING**



**PRO FILL OVERVIEW**



 **PRESSURE DRESSING**



 **FLAT FOLD TAPE**



 **QUIKCLOT HEMOSTATIC DRESSING**



 **TRAUMA GLOVES**



 **CHEST SEAL**



**1) Insert Placement**  
To begin, place the inner Trauma Kit out with the pull tab facing as shown. Then insert Medical items.



**2) Medical Items Placement**  
Stack items as shown below

**A**  **FLAT FOLD TAPE**  
**CHEST SEAL**

**B**  **TRAUMA GLOVES**

**C**  **PRESSURE DRESSING**  
**QUIKCLOT HEMOSTATIC DRESSING**

**REQUIRES FOLDING**

**3) Securing Your Medical Items**  
1. Fold Trauma Kit inner pouch  
2. Insert into Pouch







## ESSENTIALS FILL OVERVIEW



 **PRESSURE DRESSING**



 **FLAT FOLD TAPE**



 **QUIKCLOT HEMOSTATIC DRESSING**



 **HYPOTHERMIA BLANKET**



 **TRAUMA GLOVES**

When folding excess medical component packaging, take care to not fold, crease or crush the product inside.  
Please contact Customer Service if you have questions.

Specifications are approximate and subject to change without notice. ©2022 Blue Force Gear, Inc. All Rights Reserved.

**LIFETIME WARRANTY**



**1) Insert Placement**  
To begin, place the inner Trauma Kit out with the pull tab facing as shown. Then insert Medical items.



**2) Medical Items Placement**  
Stack items as shown below

**A**  FLAT FOLD TAPE  
 HYPOTHERMIA BLANKET

**B**  TRAUMA GLOVES

**C**  PRESSURE DRESSING  
 QUIKCLOT HEMOSTATIC DRESSING

**REQUIRES FOLDING**

**3) Securing Your Medical Items**  
1. Fold Trauma Kit inner pouch  
2. Insert into Pouch

