



ADVANCED FILL OVERVIEW



◆ HYPOTHERMIA BLANKET



◆ COMPRESSED GAUZE



◆ MEDICAL SHEARS



◆ FLAT FOLD TAPE



◆ TRAUMA GLOVES



◆ PRESSURE DRESSING



◆ CHEST SEAL



◆ QUIKCLOT HEMOSTATIC DRESSING



◆ NASOPHARYNGEAL AIRWAY



◆ DECOMPRESSION NEEDLE



1) Insert Placement
To begin, place the inner Trauma Kit out with the pull tab (medical insignia) facing as shown. Then insert Medical items A-E in alphabetical order.



Note: placement of shears in the corner pocket - the lip faces away from the chest seal packaging.




2) Medical Items Placement
Stack items as shown below

- A** HYPOTHERMIA BLANKET
- FLAT FOLD TAPE
- COMPRESSED GAUZE
- B** PRESSURE DRESSING
- QUIKCLOT HEMOSTATIC DRESSING
- C** CHEST SEAL
- MEDICAL SHEARS
- D** NASOPHARYNGEAL AIRWAY
- DECOMPRESSION NEEDLE
- E** TRAUMA GLOVES

REQUIRES FOLDING

Securing Your Medical Items

1. Fold Trauma Kit inner pouch
2. Insert into Pouch
3. Secure with Hook and Loop Flap

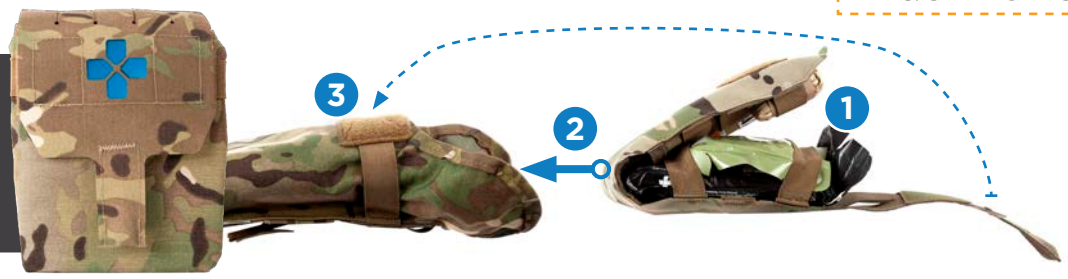

When folding excess medical component packaging, take care to not fold, crease or crush the product inside. Please contact Customer Service if you have questions.



PRO FILL OVERVIEW



◆ **HYPOTHERMIA BLANKET**



◆ **COMPRESSED GAUZE**



◆ **MEDICAL SHEARS**



◆ **FLAT FOLD TAPE**



◆ **TRAUMA GLOVES**



◆ **PRESSURE DRESSING**



◆ **CHEST SEAL**



◆ **QUIKCLOT HEMOSTATIC DRESSING**

When folding excess medical component packaging, take care to not fold, crease or crush the product inside. Please contact Customer Service if you have questions.



1) Insert Placement
To begin, place the inner Trauma Kit out with the pull tab (medical insignia) facing as shown. Then insert Medical items A-D in alphabetical order.



Note: placement of shears in the corner pocket - the lip faces away from the chest seal packaging.




2) Medical Items Placement
Stack items as shown below



REQUIRES FOLDING

Securing Your Medical Items

1. Fold Trauma Kit inner pouch
2. Insert into Pouch
3. Secure with Hook and Loop Flap

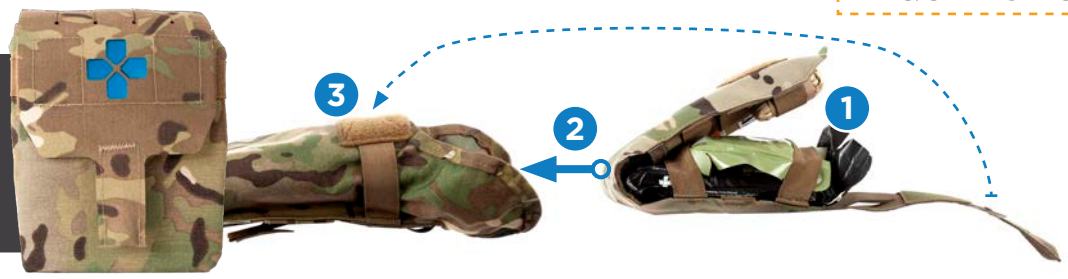



ESSENTIALS FILL OVERVIEW



◆ HYPOTHERMIA BLANKET



◆ COMPRESSED GAUZE



◆ PRESSURE DEVICE



◆ FLAT FOLD TAPE



◆ TRAUMA GLOVES



◆ PRESSURE DRESSING



◆ QUIKCLOT HEMOSTATIC DRESSING



1) Insert Placement
To begin, place the inner Trauma Kit out with the pull tab (medical insignia) facing as shown. Then insert Medical items A-D in alphabetical order.



2) Medical Items Placement
Stack items as shown below

A EMERGENCY THERMAL BLANKET → HYPOTHERMIA BLANKET
 FLAT FOLD TAPE →
 COMPRESSED GAUZE

B PRESSURE DRESSING →
 QUIKCLOT HEMOSTATIC DRESSING

C PRESSURE DEVICE

D TRAUMA GLOVES

REQUIRES FOLDING

Securing Your Medical Items

1. Fold Trauma Kit inner pouch
2. Insert into Pouch
3. Secure with Hook and Loop Flap

